NuSmile®

NuName!™ *Varnish*

5% Sodium Fluoride

INSTRUCTIONS FOR USE

Product Description

NuNamel Varnish™ is a flavored, xylitol-sweetened, 5% sodium fluoride varnish in a resin carrier delivered in a 0.4 ml unit dose blister pack.

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Form

- 0.4 ml unit dose pack
- Application brush

Indications for Use/Intended Purpose

NuNamel Varnish™ is 5% sodium fluoride in a varnish carrier which produces a mechanical occlusion of the dentinal tubules in the treatment of tooth hypersensitivity.

Contraindications

- Do not use on patients with ulcerative gingivitis or stomatitis.
- Do not use on patients with a known sensitivity to ingredients including colophony (kolophonium) and/rosin. If allergic reaction, dermatitis, or rash develops, consult a physician.
- Do not use on patients with known allergies or sensitivities to ingredients listed within the SDS. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions

- Patient, clinician, and assistant must wear protective eyewear.
- In rare instances, edematous swelling has been reported after extensive fluoride varnish treatment and dyspnea has been reported in asthmatic children. If this occurs, thoroughly brush teeth to remove the varnish film and have patient contact their medical provider.
- Some patients may experience nausea after application. If nausea persists, thoroughly brush to remove the varnish film.
- Do not use with other topical fluoride treatments the same day and instruct patient to refrain from using oral fluoride supplements for 4 (four) days after treatment.
- Do not remove foil lid from the unit dose until ready to use.

Instructions

1. Thoroughly brush teeth, rubber cup polish teeth, or wipe teeth with gauze prior to placement.
2. Open blister pack by peeling back and removing the foil lid. Using the disposable application brush, lightly stir the varnish.
3. Dry area to be treated.
4. Apply a thin, smooth layer of *NuNamel Varnish™* to as many surfaces as possible.

NOTE:

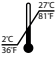
- Patients should avoid brushing and flossing for 4–6 hours after treatment. If possible, wait until the next morning.
- Patients should avoid hot foods or drinks as well as products containing alcohol (e.g., beverages, oral rinses, etc.) for 4–6 hours after treatment.
- *NuNamel Varnish™* will begin to flake off with each brushing.
- For maximum benefit, *NuNamel Varnish™* can be reapplied every 3–6 months.

Storage and Disposal

Dispose of waste according to local rules, guidelines, and regulations. Store according to outer package label.

Miscellaneous Information

Report any serious incident to NuSmile and the competent authority. www.nusmile.com | +1 713.861.0033

	Manufacturer	REF	Catalog Number	 eIFU indicator	Consult Instructions For Use
	Temperature	LOT	Lot Number		Expiration Date
	Single Use	R_{only}	Prescription Only	EN - For professional use only ES - Solo para uso profesional FR - Pour usage professionnel seulement	

 **Mfg. for NuSmile, Ltd**
3315 West 12th St
Houston, TX 77008 USA
+1 713.861.0033

nusmile.com
IFU-124 Rev-0
Made in USA from U.S. and globally sourced materials
1011806AR01 101024

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